

#### IV. Cautions.



Don't expose scale to moisture



Don't drop or mistreat the scale.



Don't immerse the scale in water



Don't overload the scale. This will permanently damage it!




Don't leave weight on the scale when not in use.



Replace batteries when the scale displays “”



American Weigh Scales, Inc.

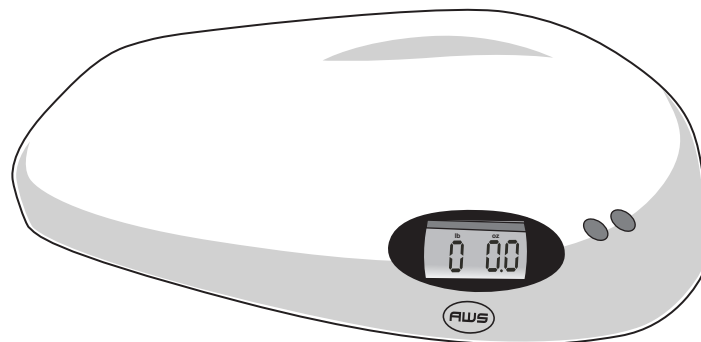


---

## **BB-44** Baby Scale

*User Manual*


## I. Operation

(1)

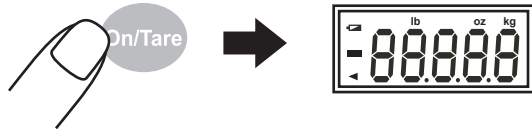


Press “ On/Tare ” to turn on the scale.

(2)

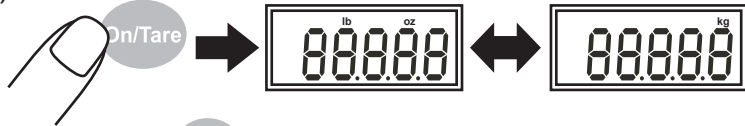

Press “ On/Tare ” to select the desired weighing unit (kg, lb:oz, oz) **before the display shows 0.00**

(3)

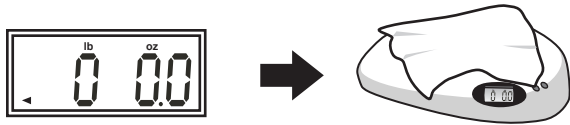


When the LCD displays “ ◀ ”, place the clothes you will be weighing your baby in on the scale.

(4)



Press “ On/Tare ” quickly to zero the scale out with the clothes on the scale. Now place your baby on the scale with the clothes

(5)

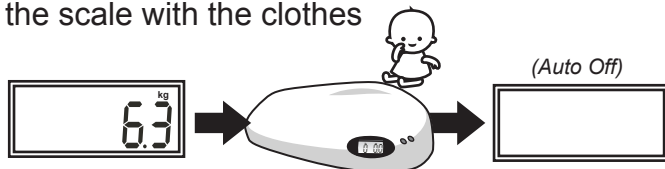


Wait for a few seconds. The weight will display on the LCD. You can now take your baby off the scale. The scale will auto-off after 60 seconds of no use.

## II. Music

This scale has a built in music feature to make the experience more enjoyable for you and your baby.

(1)

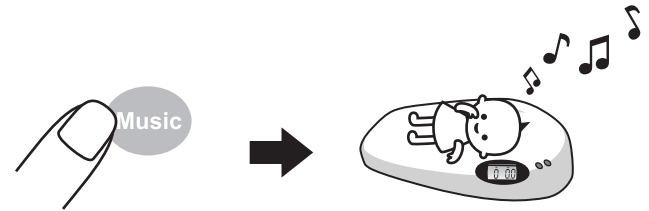

Press “ Music ” to turn the music feature ON/OFF

## III. Specifications

1. **Capacity:** 44lb / 20kg

2. **Min Capacity:** 30g

3. **Graduation:** 0.1oz / 10g

4. **Units:** kg, lb:oz, oz

5. **Over-load:** “ O-Ld ”

5. **Low Battery:**