


CBS-330

USER'S MANUAL

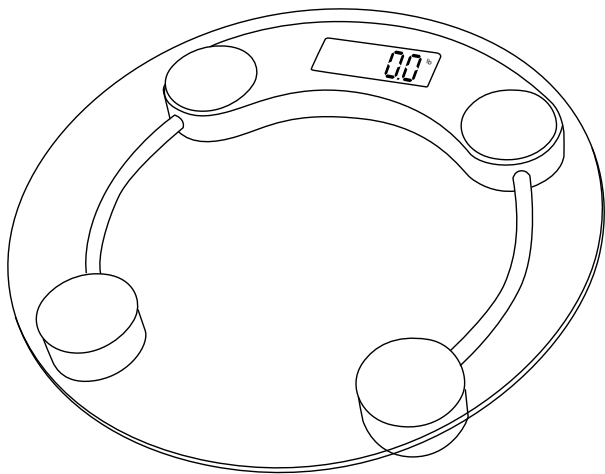

330lb x 0.2lb /
150kg x 0.1kg




AMERICAN WEIGH
SCALES INC.

American Weigh
CBS-330 User's Manual
330lb x 0.2lb

Thank you for purchasing the CBS-330 Round Bathroom Scale. With normal care and proper treatment, it will provide you with years of reliable service. Please read all operating instructions carefully before using the scale.

- Avoid any exposure to extreme heat or cold. Your scale will work better when operating in normal room temperature. Always allow the scale to acclimate to normal room temperature for at least an hour before use.
- Give the scale sufficient warm up time before use. Usually 30 to 60 seconds to give the internal components time to stabilize.
- Keep your scale in a clean environment. Dust, dirt, moisture, vibration, air currents, or a close proximity to other electronic equipment can cause an adverse effect on the reliability and accuracy of your scale.
- Handle with care. Gently apply all items to be weighed onto the weighing surface. Although this scale is designed to be quite durable, avoid rough treatment as this may permanently damage the internal sensor.
- Do not shake, drop, or otherwise shock the scale. This is a precision instrument and must be handled with extreme care.
- Only operate the scale on a stable, vibration free surface.
- Always store in a dry place to avoid moisture accumulation on internal components.


Parts Description

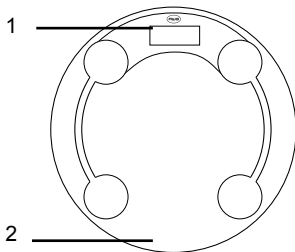

1. LCD Display

2. Tempered Glass

3. Battery Compartment
(LB/KG/OFF Switch inside)

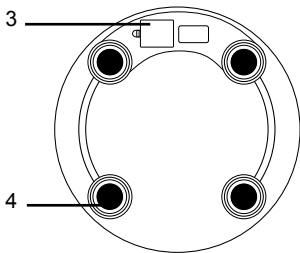

4. Scale feet

I. Setup

Remove the main unit from the package. Open the battery compartment (see above) and flip the switch to LB for pounds, or KG for kilogram. You are now ready to use your scale!

Note: You should set the switch to OFF if storing the scale for extended periods of time.

II. Weighing

1. Tap the scale once with your foot to turn it on.
2. Wait for the scale to go to “0.0” before stepping on.
3. Step onto the scale and wait for the readout to stabilize.
4. The weight will be displayed on the LCD
5. After 10 seconds, the scale will turn off (Auto-Off)
6. Replace the battery if “LO” is displayed on the LCD.

Note: For best results, ensure that you stand on the center of the scale platform. Always use the scale on a hard surface. Scale will not give accurate readings on carpeted surfaces.

III. Mode Selection (See prev. page for button locations)

1. Turn the scale over and locate the LB/KG/OFF button inside the battery compartment.
2. Flip the switch to the up position for KG, or down for LB. (Note the change on the LCD)
3. Set the scale right-side-up again when mode is selected.

IV. Specifications

Capacity:	330lb ; 150kg
Graduation:	0.2lb; 0.1kg
Units:	pounds (lb) ; kilograms (kg)
Display:	4 digits 1” in height