

330LPG

Bathroom Scale



330lb x 0.2lb
150kg x 0.1kg




330LPG Manual

Thank you for purchasing the American Weigh 330LPG Bathroom Scale. Please read all operating instructions carefully before use. Scales are precision instruments and should always be handled with proper care. To ensure years of reliable service, keep these simple tips in mind:


- Do not exceed the scales maximum capacity. Overloading your scale can permanently damage it!
- Avoid exposure to extreme heat or cold. Scales perform best at normal room temperature. If temperatures have changed dramatically, recalibration may be necessary.
- Store your scale in a clean, dry location. Dust, dirt, and moisture can accumulate on the weighing sensors and electronics causing innacuracy or malfunction.
- Avoid using your scale in close proximity to strong radio wave emitting devices such as cordless phones.
- Always weigh on a flat and level surface, free from vibrations and drafts. The corner of a room is usually the most stable.
- Avoid dropping your scale. The warranty does not cover damage due to rough treatment or overload.
- To avoid slipping, do not step on the scale with wet feet.
- Do not use the scale on carpet.

I. Display Overview

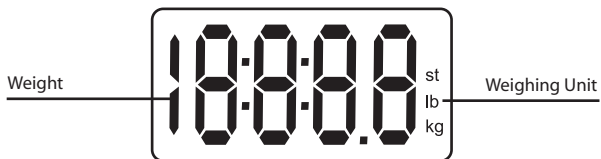

II. Battery Installation

This scale should come with the battery already installed. However, you must remove the plastic insulator strip before first use. To install the battery, turn the scale over and remove the battery cover. Insert one (1) CR2032 battery according to the diagram below. Replace the battery cover.

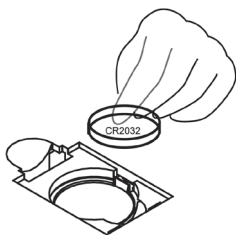

III. Turning the Scale On and Off

This scale has an Auto-On and Auto-Off feature. The scale will automatically turn on when the user steps on the platform and automatically turn off after the user steps off the platform. This scale is ready to weigh, and does not require tapping first before stepping on the scale.

IV. Changing Weighing Units

To change the current weighing unit, use the UNIT button on the bottom side of the scale. Notice the display will change between kg, st:lb, and lb with each button press.



V. Weighing

- Once you have chosen the desired weighing unit (see above), step onto the scale to activate the Auto-On feature.
- The display will begin to flash, then lock the weight on the display once the weight is stable.
- The scale will turn off automatically after 10 seconds.

Notes: Do not leave items on the scale when not in use. If moving the scale around, please first allow the display to

turn off once before stepping on. This will allow the zero-point to be re-established. Otherwise the weight of the scale may be reflected in your results.

VI. Troubleshooting

Problem: The display shows “- - - -”:

Solution: The scale is overloaded. Remove the excess weight.

Problem: The display shows “Lo”:

Solution: Battery power is low. Replace the batteries.

Model	330LPG
Capacity	150kg / 330lb / 23st:13lb
Resolution	0.1kg / 0.2lb / 0.2lb (1.0lb for ≥ 280 lb)
Weighing Units	kg, lb, st:lb
Scale Dimensions (inches)	11.8 x 11.8 x 0.7”
Power	1 x CR2032 Battery (included)
Warranty	10 Year Warranty

American Weigh Scales, Inc.

2 Carriage Ln, Suite 2

Charleston, SC 29407

Tel: 866-643-3444 (toll free)

Fax 843-225-7284

